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**EN Catheter for neonates of low birth weight  
 CN-01 100 cm**

**Catalogue No.:** V686417-01ND  
**Product type:** medical device  
**Design:** sterile, ethylene oxide sterilization  
**Target patient:** newborn (neonatal patient)  
**Designated user:** medical personnel with appropriate education, knowledge and practical experience in medical devices.  
**Materials used:** PVC  
**Compatibility:** With unthreaded or threaded disposable syringe (see Manufacturer's Note). CN-01 100 cm is not intended for administration of drugs.  
**Contraindication:** Do not use on any patient with a known allergic reaction to any of the product components.  
**Adverse effects:** No adverse effects are known if the set is used in accordance with the instructions specified in the Warning and Use sections.  
**Warning:** While feeding a baby from a feeding bottle, the bottle should be in level with the baby's head.  
 Do not use if the sterile packing is opened or damaged, or when the protective covers dropped off. Do not use after the expiry date.  
 Repeated use or repeated sterilization creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability, may lead to injury, illness or death of the patient.  
**Storage:** The products should be stored in dry, ventilated, dust-free and dark rooms without direct sunlight to avoid their damage and deformation.  
 Products must be stored in the packaging as delivered by the producer. Products must be stored at temperatures from +5°C to +25°C and the relative air humidity is max. 75%.

**Intended use**

CN-01 100 cm is utilised to provide supplementary food to a baby that is breastfed, but temporarily receives less milk from the breast than he/she needs. The catheter enables the baby to suck breast milk from his/her mother's breast and, at the same time, supplementary food from a feeding bottle or a syringe. This method perfectly maintains proper way of sucking; mother's hypophysis is stimulated; hormone prolactin is created, and, as a result, milk is produced.

**Instruction for use**

- Take the catheter out of the packaging under aseptic condition.
- The supplementary food can be supplied from a feeding bottle or a syringe:
  - feeding from a feeding bottle
    - cut the cap (cover) of the catheter connecting beam off
    - adapt the feeding bottle teat – cut it to create an opening allowing the adapted end of catheter to be pulled through (= the catheter connecting beam with a cut-off cap)
    - fill the feeding bottle with appropriate supplementary food recommended by the physician, and attach the teat to the bottle
    - pull the adapted catheter end through the opening in the teat, and dip it in the food
  - feeding from a syringe
    - open the cap (cover) of the catheter connecting beam
    - connect the syringe with supplementary food to the catheter connecting beam
- Position the baby at the breast, and insert the rounded end of the catheter to the baby's mouth so that he/she accepts both the papilla and the hose.

**Manufacturer's note:** The manufacturer warns the user of the existing possibility of incorrect connection. The connection connector used on the catheter is a separately developed alternative connector. Compatibility with syringe has been verified by the manufacturer in physical and functional tests.

**PROCEDURE FOR SAFE DISPOSAL OF MEDICAL DEVICES AFTER USE:**

Put the medical device in a blue PE bag intended for hazardous - infectious waste. After filling, seal the labeled bag (waste code, place of origin, date and signature of the responsible person) and transport it safely to the incinerator = decontamination device. Removal of the dangerous property of infectivity means proper decontamination. Waste sorting is carried out according to the operating rules of the medical facility.

**NOTE TO USERS:** Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.

**Used symbols on packaging**

REF	Catalogue Number	Keep away from Rain	LOT	Batch Number	Manufacturer
Use by / Expiry date		Temperature Limitation	Read instructions for use	STERILE EO	Sterilised by Ethylene Oxide
Pyrogen free		Keep away from sunlight	Latex free product	Does not contain DEHP phthalate	
Do not Resterilize		For single use only	Do not use if unit package is damaged	MD	Medical Device
Date of Manufacture					

The instruction for use is also available in electronic form at [www.gama.cz](http://www.gama.cz). Free copies of other copies are available on request via e-mail: [sales@gama.cz](mailto:sales@gama.cz).

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